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## Dermatix® Silicone gel for scar reduction

### What is Dermatrix gel?

Dermatix is a topical silicone gel that is transparent, dries quickly and helps maintain the skin's moisture balance, while aiding the management of scarred skin surfaces resulting from surgery, burns, and other injuries. Dermatrix has been shown to flatten, soften and smooth scars, relieve the itching and discomfort of scars as well as reduce the discoloration associated with scars. Dermatrix gel is easily applied to all areas of the skin, including the face, joints and flexures, and dries rapidly to form an invisible "sheet".

Dermatix was developed using the same basic long chain polymers as topical silicone gel sheeting, but it is available in a tube. It contains polysiloxane. The gel is available in tubes of 15 g and 60g.\*

### What is Dermatrix gel used for?

Dermatix is used for the treatment and prevention of keloids and hypertrophic scars (such as those resulting from general surgical procedures, trauma wounds and burns), after the wound has healed and the skin surface is intact.

### How should Dermatrix gel be used?

Cut the tip of the tube. Wash the affected area and pat dry. Dermatrix should only be applied to clean, dry skin. Gently massage a small amount of Dermatrix into the scar. If you apply too much gently wipe away any excess with a tissue in order to avoid staining of clothing. Apply once in the morning and once at night. Once dry, Dermatrix may be covered with cosmetic blush. Dermatrix can also be used on children.

### When should Dermatrix gel not be used?

- Dermatrix should not be applied to open or fresh wounds.
- Dermatrix should not be placed in contact with mucous membranes, or applied too close to the eyes.
- Dermatrix should not be applied over other skin treatments without the advice of your doctor.

### Are there any possible unwanted effects of Dermatrix gel?

Rarely, the gel may cause redness, pain or irritation to the skin. If you are concerned about these or any other unwanted effects, consult your pharmacist or doctor.

### How should Dermatrix be stored?

- Keep out of reach of children.
- Store below 25°C and away from heat sources.
- Do not use after the expiry date (☞) printed on the pack.

### Further information

You can get more information about Dermatrix from your pharmacist or doctor.

#### Hanson Medical, Inc., Kingston, WA 98346 USA

 Meda Pharmaceuticals Ltd, Skyway House, Parsonage Road, Takeley, Bishop's Stortford, CM22 6PU, UK

### Distributor in UK:

Meda Pharmaceuticals Ltd, Skyway House, Parsonage Road  
Takeley, Bishop's Stortford, CM22 6PU

### Distributor in Ireland:

Meda Health Sales Ireland Ltd, Office 10  
Dunboyne Business Park, Dunboyne, Co Meath

\*Not all countries supply all presentations

**MEDA SPECIFICATION BOX LEAFLET**

<b>COUNTRY</b>	BE / NL / FR / UK / IE	<b>COMPONENT SIZE</b>		<b> FONTS USED</b>
<b>PRODUCT</b>	DERMATIX GEL	<b>PHARMACODE</b>		Myriad Pro Regular
<b>PACK SIZE</b>	15 g	<b>EAN CODE</b>		<b>Myriad Pro Bold</b>
<b>MEDA ITEM CODE</b>		<b>FONT SIZE</b>		
<b>MATERIAL NR</b>		<b>COLOURS USED</b>		
<b>DESIGNED BY</b>	Olivia Veltren	 BLACK		
<b>DATE</b>	25.03.2009			
<b>COMMENTS</b>	New Layout			<b>1. AK 07.04.2009</b> <b>2. AK 20.05.2009</b> <b>3. AK 24.06.2009</b> <b>4. AK 14.07.2009</b> <b>5. AK 06.08.2009</b> <b>6. AK</b>
<b>APPROVAL</b>	<b>SIGNATURE</b>			<b>DATE</b>